



LAB instructions ↓

Prepare the received Capitainer®B sampling card as below.



1 Peel-off tabs on the backside of the card.



2 Tear the peel-off tabs.



3 Insert a sharp tool, e.g. tweezers under the paper disc.



4 Carefully remove the paper disc.



5 Put the disc in a preferred vial, e.g. microtiter plate for subsequent elution.



6 qDBS sample ready for analysis.



Capitainer AB  
Norr Mälarstrand 54  
SE-112 20 Stockholm SWEDEN  
info@capitainer.se

capitainer.se